

# Peak Go Seat


Product Code: 9346376060523

## Specifications

Product Specifications	
Product Height	90cm
Sitting Height	60 cm
Product Weight	2.5 kg
Weight Capacity	100 kg
Material (Trolley)	Aluminum + Plastic
Material (bag)	Nylon
Color	Green

Peak Care Equipment  
149 Princes Highway  
Unanderra NSW 2526  
Tel: +612 4272 2688  
[www.peak-care.com.au](http://www.peak-care.com.au)

Distributor: 1300 848 077

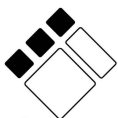

**back to sleep**  
the bedding & mobility specialists



User Manual

# Thank You!

Thank you for purchasing the Peak Go Seat Cart.

The Peak Go Seat is ideal for outdoor use. Kindly take time to go through this manual which gives you more details about your Peak Go Seat. Kindly keep this booklet handy and use it as and when required.


Peak Go Seat is warranted to be free of defects in material and workmanship for twelve months from the date of purchase.

This device was built to exacting standards and carefully inspected prior to packing and shipping. This warranty is an expression of our confidence in the materials, our workmanship and an assurance of the same to our customers.


In the event of a defect covered by this warranty, we will at our option supply parts or replace the unit. This warranty does not cover device failure due to owner's misuse or negligence or normal wear and tear. This warranty does not extend to nondurable components such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries, please contact the distributor.


## How to install

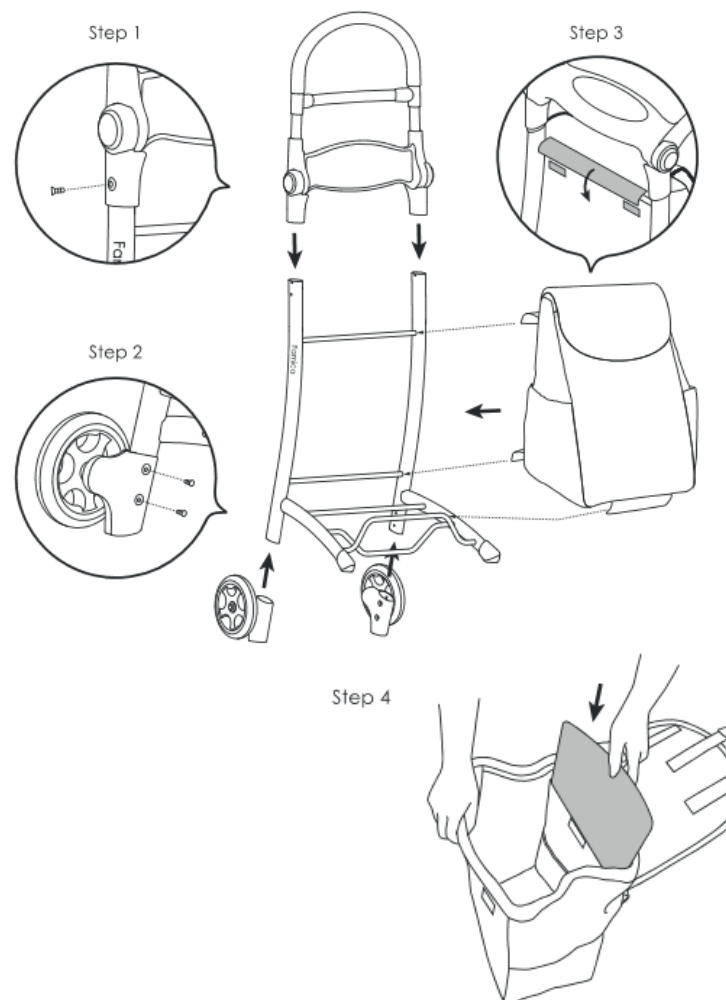


### How to use

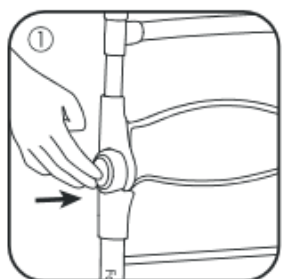

Press the button

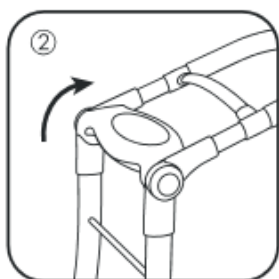

Pivot to seat

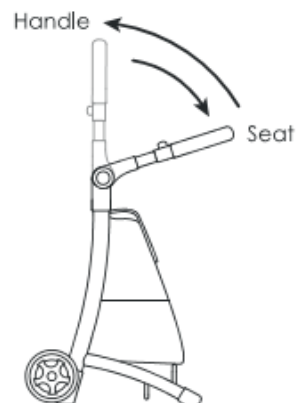

### CARE AND MAINTENANCE

- The Peak Go Seat can be cleaned by using mild soap and water. Wipe dry after cleaning.
- Periodically inspect the Peak Go Seat for loose screws or wheels. Please do not use if any screws or wheels have become loose.

### SAFETY INSTRUCTIONS

- Do not use the Peak Go Seat as seat on slippery surfaces.
- Do not stand on the handle seat.
- Do not stand on handle whilst it's in the seated position.
- Check all screws are tight before using for the first time.
- Always keep the feet flat on the floor when sitting.
- Storage weight limit: 30kg.
- Seat user maximum weight: 135kg.
- Do not propel the Peak Go Seat whilst seated.