



A & E Upholstery Pty Ltd
trading as **Age Prepcare**



ABN 52 608 234 761

P: 03 9702 3544

E:
aeuph@bigpond.com.au

17-19 Star Crescent Hallam VIC 3803

E: Info@ageprepcare.com.au

The quality construction of a lift mechanism using the latest technology, bring restful comfort to those who need help to sit and stand. The addition of a recliner chair mechanism also allows the lift motor to recliner the chair to the desired comfort position. This is a blessing for all people, especially for the elderly. This uniquely designed chair with the integral footrest offers total support throughout its varied positions, and the simple-operated hand control can be used to change positions smoothly and easily. For complete comfort, our full adjustable rising and reclining chairs will give you total support whether you're stretching out, sitting up or simply laying back and relaxing.

Operating Instructions and Attentions:

- Before connecting your electric lift chair to the power supply, please ensure that the power cord, hand control cord and hand control are free of the action, and the power cord and hand control cord have not wrapped around the mechanism during transport.
- After you have switched on the power supply at the wall, the chair is ready to be used. There are no other "ON" or "POWER" buttons.
- To raise chair, please push down on the top button on the hand control. The chair will raise from the floor, tilt forward and stop at a set height. When the chair is rising and tilting forward, the user should step on the floor securely and beware slipping out from the chair.
- To lower the chair, please push down the bottom button on the hand control. The chair will lower to the floor. Please ensure that no animals or persons are under the chair when operating.
- To recliner chair, when the chair is in the lowered position, please push down the bottom button on the hand control. The footrest will extend. The chair back will recliner and please the enough spaces for the reclining. Keep finger on the button until desired recline position is reached.
- To close recline, please push down on the top button on the hand control until the footrest returns to the closed position. Beware of the footrest to clamp anything or anybody when the footrest is closing. Continued depression of the button will cause the chair to move to the raised position.
- When the chair comes to a stop at either the fully raised of fully reclined position, please release the button or the overload may shut the power supply down prematurely.
- The chair can stop at any position by releasing the button.

Lifting Capacity:

The frame and structure of the lift chair have been reinforced to ensure maximum strength and stable ride. The chair can lift MAXIMUM load of 160kgs.

Important Safety Instructions:

The motor is protected by thermal overloads. If the chair is overused during a short space of time, it will automatically shut down, and you should wait until the motor cools before father use.

The eliminates any risk of overheating and deters children playing but please remember that the lift chair is not a toy and should be used as a plaything by children. Warranty will be voided in the case of incorrect operation of customer.

Warnings:

- This chair is suitable for use for personas not exceeding 160kgs.
- Do not place overpower cords or cause power cords to go inside this chair at any time.
- Do not allow children to play on this mechanized furniture.
- Ensure all persons and animals are clear of chair when operating.
- Do not allow any liquids to come in contact with hand control.